Monopolare Saugrohre Monopolar Suction Tubes

REF:

71 50 00 – 71 50 49

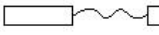
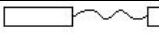
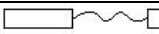
Sutter Medizintechnik GmbH
Tullastr. 87
D-79108 Freiburg/Germany
Tel.: (+49)-761-515510
Fax: (+49)-761-5155130
E-Mail: info@sutter-med.de
www.sutter-med.de

REF 89 90 10_01

08/2008 © Sutter Medizintechnik GmbH

TAB 1		REF
A	 -> Sutter BM-780 II, Martin, Berchtold, Aesculap, ErbeT-Serie	36 01 85
B	 -> Erbe ACC /IC / ViO, Karl Storz	36 01 86
C	 -> Sutter CURIS®, Valleylab, Conmed, Bovie, Bowa	36 01 87
D		90 09 60

Deutsch

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© STERRAD® is a trademark of Johnson & Johnson, Inc.

[1] deconex 28 ALKA ONE (Borer Chemie AG)

English

Product / User / Disposal:

Electrosurgical accessories may only be used and disposed of by trained medical personnel.

These instructions do not substitute the instructions for the electrosurgical unit and other accessories used.

 **Non-sterile.** Clean and sterilize before first and subsequent uses.

Intended use:

Monopolar suction tube for the aspiration of smoke and liquids and for coagulation of soft tissue during surgical procedures.

Product life:

The life cycle of the product depends on the use and treatment it gets.

Prior to use:

Examine the integrity of the insulation as well as the cleanness and integrity of the instrument.

Test the insulation with an appropriate testing device.

Never use damaged products.

 Only use safe and sterilized products.

Connect the monopolar suction tube to the correct cable from Sutter [TAB1:A-C].

A certain discolouring at the instrument tips is normal and harmless.

Connect instrument and cable only to an electrosurgical unit that has been switched off or is in standby mode.

 Disregarding these instructions may lead to burns and electrical shock.

Ensure that patient plate is properly attached.

During use:

Always use the lowest power setting possible to achieve the desired surgical effect.

 The instrument tip may be hot after use and cause burns.

 Never place the instrument on or close to the patient.

 Do not use near inflammable or explosive substances.

We recommend the use of non-conductive liquid when using the monopolar suction tube together with irrigation instruments.

 Max. voltage 6000 Vpp.

Cleaning and sterilization:

Observe nationally applicable regulations and guidelines.

Disconnect the instrument from the cable and tube.

Do not allow blood and debris to dry on the instrument.

Use a soft cloth or brush [TAB1:D] to remove blood and debris.

Do not use aggressive/abrasive cleaners.

Pre-clean and rinse thoroughly with clean water.

Do not put in hydrogen peroxide (H₂O₂)!

Machine-clean and disinfect the instrument.

Make sure that openings are not congested. If necessary, use a cleaning wire or brush [TAB1:D].

Machine cleaning/disinfection:

Thermal disinfection for min. 5 minutes at >90 °C (>194 °F).

We recommend cleaners suitable for plastic materials [1].

Use an approved cleaning program. Follow the

manufacturer's instructions for dosage and the program run.

Use distilled or fully desalinated water for final rinsing cycle.

Sterilization:

Only sterilize clean/disinfected instruments.

Use paper/foil wrapping or appropriate sterilization containers.

Steam-sterilize in the autoclave.

Fractionated pre-vacuum sterilization:

Temperature 134 °C (273 °F), 3 min.; max. Temperature

138 °C (280 °F), max. duration 20 minutes.

Ensure that the instrument is dry after sterilization!

Do not sterilize in hot air.

Do not sterilize in STERRAD®.

 Destroy instruments that might have been contaminated with prions (CJD). - Do not reuse!

Storage/transport:

Store in a cool and dry place. Do not expose to direct sunlight. Store and transport in safe containers/packaging.

For returns only cleaned and disinfected instruments in sterile packaging will be accepted.

Sutter Medizintechnik cannot be held liable for changes to the product after purchase or deviations from these instructions. Contact the manufacturer to obtain information on applicable standards, regulations and existing validations.

Subject to change.

STERRAD® is a trademark of Johnson & Johnson, Inc.

[1] deconex 28 ALKA ONE (Borer Chemie AG)